



High Risk Nitrile Examination Gloves



Glove of Choice for First Responders, EMS/Police, Health Care/First Aid, Chemotherapy/Home Infusion, Compounding

- Meets **ASTM D6978** for Resistance to Permeation by **15 Chemotherapy Drugs**
- Meets **ASTM D6978** for Resistance to Permeation by **Fentanyl**
- Meets **ASTM D6978** for Resistance to Permeation by **Xylazine**
- Use for **USP800 Compliance**
- **High Risk** 8mil, 295mm/11.6" Extended Cuff
- **0% Viral Penetration**
- **0% Alcohol Permeation**
- **Low Dermatitis** Patented Technology
- Packaging in English and Spanish

Tested for Use with **15** Chemotherapy Drugs



Superior Permeation Protection

- Meets **ASTM D6978** for Resistance to
 - Permeation by **Chemotherapy Drugs**
 - Permeation by **Fentanyl** and **Xylazine**

Minimum Breakthrough Detection Time in Minutes

Chemotherapy Drug & Concentration	
> 240 minutes	Cisplatin 1.0 mg/ml, Cyclophosphamide (Cytosan) 20 mg/ml, Cytarabine 100 mg/ml, Dacarbazine (DTIC) 10.0 mg/ml, Doxorubicin Hydrochloride 2.0 mg/ml, Etoposide 20.0 mg/ml, Fluorouracil 50.0 mg/ml, Ifosfamide 50.0 mg/ml, Methotrexate 25 mg/ml, Mitomycin C 0.5 mg/ml, Mitoxantrone 2.0 mg/ml, Paclitaxel (Taxol) 6.0 mg/ml, Vincristine Sulfate 1.0 mg/ml
Not Recommended for Use	Carmustine (BCNU) (3.3 mg/ml), Thiotepea (10.0 mg/ml)
Fentanyl Citrate & Concentration	Gastric Acid
> 240 minutes	> 240 minutes
Fentanyl Citrate Injection (100mcg/2ml)	Simulated Gastric Acid Fluid
Xylazine HCl	Made in house with reagent grade solvents; 0.2% (w/v) Sodium Chloride in 0.7% (v/v) Hydrochloric Acid
> 240 minutes	Xylazine (100mg/ml)

High Risk Nitrile Certification



Meets ASTM D6978



Use for USP800



Patented Technology*

LOW DERMA Formula



MH60153

UltiMates High Risk Nitrile Examination Gloves - Product Numbers

Item Description	Shepard Item#	McKesson Econo#	Cardinal CIN#	Cencora ABC8#	Smith Drug#
50ct UltiMates Nitrile MD	05632080		5650932		
50ct UltiMates Nitrile LG	05633080		5650940		
50ct UltiMates Nitrile XL	05634080		5650957		

